

STANDAID DELUXE SLINGS


HOYER

FITTING INSTRUCTIONS
TECHNICAL SPECIFICATIONS
WASHING INSTRUCTIONS


OXFORD

USER GUIDE FOR THE STANDAID DELUXE SLINGS

The Deluxe Standing Sling is an easy fit sling suitable for standing and supporting. It is not a general purpose sling and clients should be assessed as to suitability. The Deluxe Transport Sling is an easy fit general purpose sling giving full support to clients. It will not suit double amputees or lifting from a horizontal position.

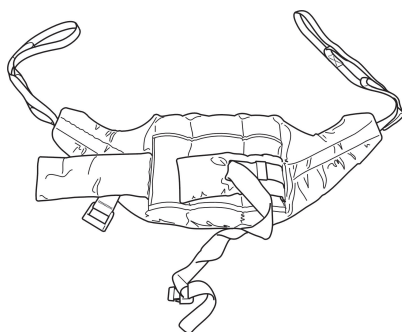

Deluxe Standing Sling

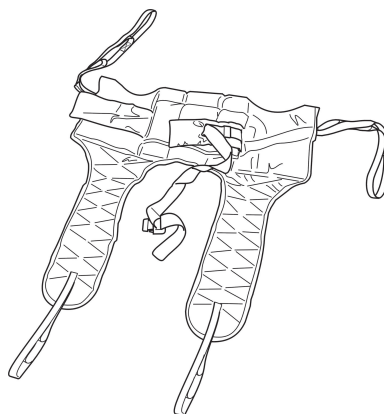


Deluxe Transport Sling

ADDITIONAL SLINGS ARE AVAILABLE TO SUIT INDIVIDUAL NEEDS.
YOU ARE ADVISED THAT YOU SHOULD ALWAYS SEEK THE ADVICE OF AN
AUTHORISED DEALER BEFORE PURCHASING.


WE WILL BE PLEASED TO ADVISE YOU AS TO YOUR NEAREST AUTHORISED DEALER.

 **Joerns**
Healthcare

HOW TO FIT THE DELUXE STANDING SLING

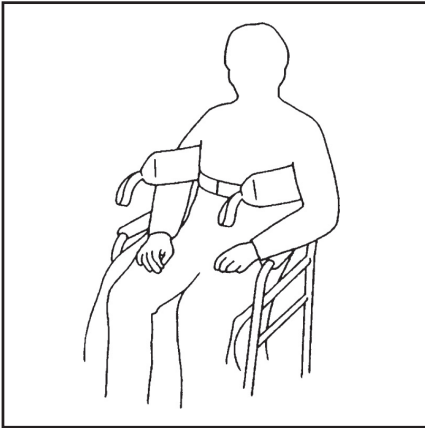

Place sling down behind client's back between base of shoulder blades and bottom of rib cage. Attach strap around the front of the client. This strap need not be tight. It is there to hold the sling in position whilst attaching the sling to the Standard. Ensure that the client's arms are outside the sling.

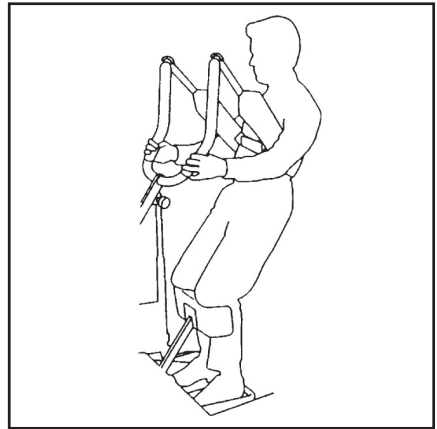

Position Standard so that the client's feet fit in the foot trays and knees rest against the knee pad. Initially attach the longer loops of the sling to the white hooks at the end of the hand grips. The client's hands should be placed on the black hand grips or support handles. *(Once the client is used to the lift, the shorter loops may be used to achieve a more upright position)*. Then push button on hand control to raise the client to the standing position.

N.B. This sling is not a general purpose sling and will not be suitable for some clients.


TIPS FOR USE

CLIENT SHOULD BE REQUESTED TO LEAN BACK INTO THE SLING FOR THE DURATION OF THE LIFT.


TALLER CLIENTS MAY NEED TO USE SHORTER LOOPS ON THE SLING FROM THE START.

SHOULD THE CLIENT START TO SLIP SO THAT THE LIFT IS COMING FROM BENEATH THE ARMPITS, THE TRANSPORT SLING SHOULD BE USED.


HOW TO FIT THE DELUXE TRANSPORT SLING

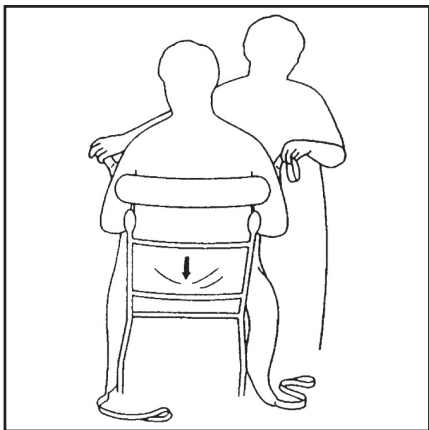


Feed the sling down the back of the client leaving the top of the commode aperture at the base of the spine. Check the sling is square across the client's base.

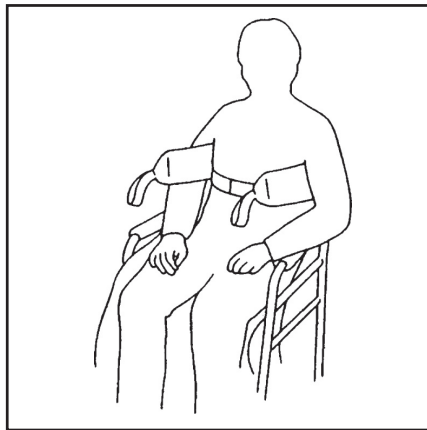

Attach the strap around the chest of the client. This strap need not be tight, it is there to hold the sling in position whilst attaching the sling to the Standardid. Ensure client's arms are on the outside of the sling.

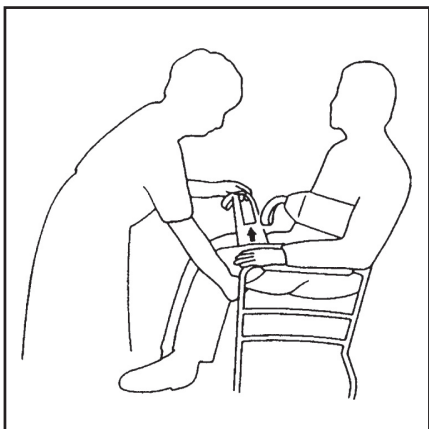

Raise the client's leg and feed the leg strap under, and up between the legs. Ensure that the sling is not twisted or creased under the thigh. Repeat this procedure for the other leg. Attach the loops on the top of the sling to the white hooks at end of hand grips.

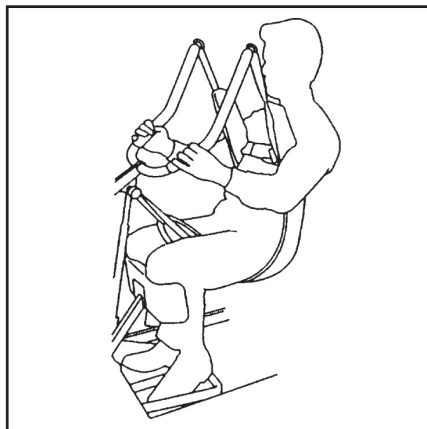

Without crossing loops on the leg straps, attach to left and right black knobs on the lifting fork. You would normally use the middle of the three loops. You are then ready to raise the client from the seat and transport. Once clear of the seat, the client will be more comfortable if lowered back to sitting position.


ENSURE THAT YOUR CLIENT'S HEAD IS SUPPORTED AT ALL TIMES

STANDAID DELUXE SLING TECHNICAL SPECIFICATIONS

STANDARD	BS EN ISO 10535
SAFE WORKING LOAD	375lbs / 170kg

SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

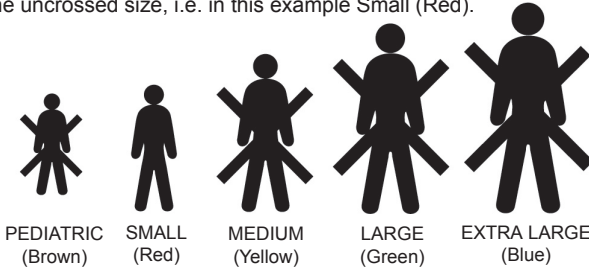

WASHING INSTRUCTIONS

Each sling is marked with the following washing instructions:

DO NOT WASH WITH BLEACH

Bleach will damage material. Slings can suffer damage during washing and drying. Check sling before each use. Bleached, torn, cut, frayed or broken slings are unsafe and could result in serious injury or death to patient. Destroy and discard worn slings. Do not alter slings. Machine wash warm. Cool tumble dry, air dry or dry at very low temperature.



ORDER CODE		US	EU	Deluxe Standing Sling (Polyester)	Deluxe Transport Sling (Polyester)
		PEDIATRIC (BROWN)	NA	SL	Special
	SMALL (RED)	NA	SL	13499N	13502N
	MEDIUM (YELLOW)	NA	SL	13500N	13501N
	LARGE (GREEN)	NA	SL	13498N	13503N
	EXTRA LARGE (BLUE)	NA	SL	Special	Special

THESE SLINGS ARE SUITABLE FOR USE WITH THE OXFORD/HOYER STANDAID



Joerns Healthcare Inc.
5001 Joerns Drive, Stevens Point,
Wisconsin, 54481-5040
Tel: (715) 341 3600
(800) 826 0270 (Toll Free)
www.joerns.com



Joerns Healthcare Limited
High Street Wollaston
West Midlands DY8 4PS England
Tel: 01384 44 66 22
Fax: 01384 44 66 01
www.joerns.com

Part no. 374-90025
Rev B